




Treating cavities
with Carisolv™

– less pain and
discomfort

Facts about Carisolv™

- Carisolv™ is a method for the gentle treatment of cavities.
- A gel softens the caries and saves as much as possible of your tooth.
- Causes less pain and discomfort.
- Reduces or eliminates the need for the drill and local anaesthetics.
- A premium form of treatment, may call for extra time and may cost somewhat more.

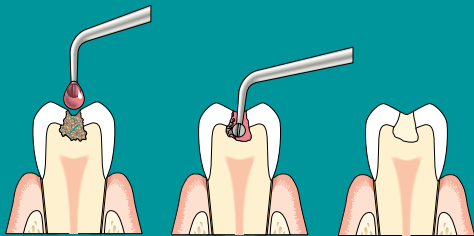

Apply

Scrape

Repair

The treatment procedure

At the outset, your dentist may need to use the drill to remove hard substance, such as enamel or old fillings. This is a quick procedure with limited discomfort. Clinical studies show, in fact, that local anaesthetics could be avoided.

The dentist covers the cavity with Carisolv™ gel, waits for 30 seconds for the carious dentine to soften and then starts carefully to scrape away the softened caries. Carisolv™ gel may be applied again and the gentle scraping repeated until the cavity is caries free. Finally, your dentist can inspect and fill it like any other cavity.

When can Carisolv™ be used?

Carisolv™ can be used in virtually any case of tooth decay. So far more than 100,000 patients have been treated. Your dentist, however, knows when it is most advantageous and is likely to recommend it in the following situations:

- Cavities on the root surface
- Deep cavities
- Children
- Patients with anxiety
- Whenever local anaesthetics are not an option

Carisolv™ is a product from the research-driven dental company MediTeam. The Carisolv™ trademark is owned by MediTeam Dental AB.

For more information:
www.carisolv.com